



DEMO GUIDE

Bite Registration

Zhermack 
Dental



The objective of this Demo Guide is to deliver information that will help you performing a better product demonstration.

Discover more on Zhermack products:



Occlusal technique

The occlusal technique consists in positioning the material on the patient's occlusal plane.



1

Occlusal technique

PREPARE THE 1:1 DISPENSER



Push the black lever underneath the rack upwards and pull the dispenser rack as far as it will go.



Lift the cartridge lock and insert the cartridge. It must be inserted with the "V" on its base facing downwards. The lock can be lowered easily.



Dispense a small amount of the two components until they flow out at the same time. Remove any excess, without contaminating the two outlet holes.



Select the appropriate mixing tip. Its color must be the same as the color of the cartridge cap.

2

Occlusal technique

PLACE THE MATERIAL ON THE OCCLUSAL PLANE AND REGISTER THE OCCLUSION



Apply the material on the occlusal plane of the model's lower jaw, observing the working time.



Once the material has been extruded, leave the used mixing tip on the cartridge as a cap until the following use.



Close the model's upper jaw on the lower one to register the occlusion.



Wait for the material polymerization to complete, before removing the bite registration.



Vestibular technique

The vestibular technique consists in positioning the material vestibularly in the patient's mouth.



1

Vestibular technique

PREPARE THE 1:1 DISPENSER



Push the black lever underneath the rack upwards and pull the dispenser rack as far as it will go.



Lift the cartridge lock and insert the cartridge. It must be inserted with the "V" on its base facing downwards. The lock can be lowered easily.



Dispense a small amount of the two components until they flow out at the same time. Remove any excess, without contaminating the two outlet holes.



Select the appropriate mixing tip. Its color must be the same as the color of the cartridge cap.

2

Vestibular technique

PLACE THE MATERIAL VESTIBULARLY AND REGISTER THE OCCLUSION



Keep the model's jaws closed and apply the material vestibularly, observing the working time.



Once the material has been extruded, leave the used mixing tip on the cartridge as a cap until the following use.



Remind that the unset material won't drip since it is thixotropic.



Wait for the material polymerization to complete, before removing the bite registration.

Bite registration

STEP BY STEP

Prepare the 1:1 dispenser



Occlusal technique

Place the material on the occlusal plane and register the occlusion.



Vestibular technique

Place the material vestibularly and register the occlusion.





Fulfilling your needs